

Patient Alert PA Fall Prevention System

QUICK START GUIDE

Emergency Caller Products GETS HELP FAST!

Emergency Caller Products (T) 800-227-2474 (F) 800-762-7613
www.emergencycaller.com sales@emergencycaller.com
U.S.A. and World Patents Issued and Pending on Certain Products



Step 2. Position the Alert Unit.

With the Patient Alert still in the off position, attach the unit to the bed-rail, to the back of the bed or to a leg of the bed using the Mesh Bag provided. It is recommended to mount the unit facing away from the patient as Patient Alert sound is designed to alert caretaker while alerting patient to return to bed.




Mesh Bag Included



Follow these simple instructions to set up your system

Step 1. Connect the cord.

Insert the connector plug on the end of the Sensor Pad into the socket on top of the Patient Alert unit as shown.

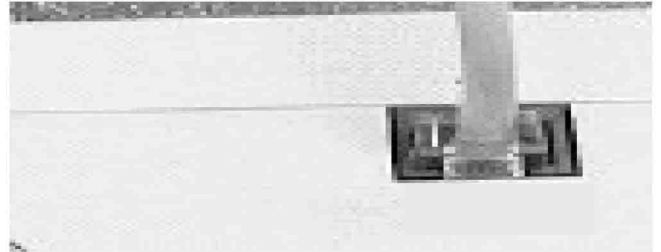

Step 3. Operate the system.

Here we are at the last step which enables Instant Activation Mode. Place the patient on the bed or chair. Now you are ready to activate the Patient Alert unit.

Step 4. Activate the System.

A- Momentary Activation (MOM)

Switch Alert unit to **MOM** position. This will activate Alarm when patient attempts exit bed. The Patient Alert unit will Automatically reset and silence alarm when patient returns to Sensor Pad position or Alert unit is turned off by caretaker.

B- Continuous Activation (CONT)

When the Alert switch is placed in **CONT** position and patient exits Sensor Pad, Patient Alert will sound alarm continuously until turned off by caretaker.

*IMPORTANT

Replace Battery If Patient Alert System Fails.
Replace Battery Periodically

Test System Before Each Use

**PLEASE SEE REVERSE SIDE
FOR LIFETIME SENSOR PAD
INSTALLATION INSTRUCTIONS**